SDI poladay CP

35% carbamide peroxide
advanced tooth brightening system

INSTRUCTIONS FOR USE

ENGLISH

Pola Day CP is a 35% carbamide peroxide, high viscosity, neutral pH in-office tooth brightening gel. It is a pleasantly flavoured, crystal clear sustained release gel with a water content high enough to reduce sensitivity.

INDICATIONS FOR USE:

- Lightly etching away surface staining on vital teeth
- Lightly etching away surface staining on non-vital teeth.
- Assisting in the brightening of discoloured teeth
- Removal of surface stains and other discoloration on teeth caused by metabolic, pharmacological, congenital, iatrogenic, traumatic, or systemic factors including:
 - Medications, such as tetracycline, minocycline
 - Fluorosis
 - Stains caused by trauma

COMPOSITION: 35% Carbamide peroxide
40% Water
0.5% Flavour
24.5% Thickening agents

NET WEIGHT:

Depending on kit purchased, 1.3g syringes or 3g syringes. Please refer to outside of box.

HYDROGEN PEROXIDE EQUIVALENT:

Pola Day CP 35% Carbamide Peroxide = 11.7% Hydrogen Peroxide

INSTRUCTIONS:

- Determine and record pre-operative shade. Please follow this Vita* shade guide arrangement according to degree of brightness: B1•A1•B2•D2•A2•C1•C2•D4•A3•D3•B3•A3.5•B4•C3•A4•C4
- Clean teeth.
- Remove the syringe cap and insert an application tip by twisting it securely onto the syringe.
- Place a small drop of gel into every compartment of the tray for the teeth undergoing treatment.
- Seat the tray, with the gel around the teeth.
- Wipe away excess gel in mouth with a tissue or dry soft brush.
- After 15 minutes of treatment, remove tray. Rinse tray and mouth with lukewarm water. NOTE: Treatment time can be up to 30 minutes if required.
- Record the final shade.

OPTIONAL FOLLOW UP TREATMENT: If Pola Day CP is used as a jump start treatment, in-office brightening treatment such as with Pola Office or Pola Office+ can be used, or take home Pola Day (3% and 6% hydrogen peroxide) or Pola Night (10% and 16% carbamide peroxide) can be prescribed.

STORAGE AND HANDLING:

- Store product in the refrigerator (2°-8°C / 35°-45°F).
- Do not use after expiration date.
- Use product at room temperature.

PRECAUTIONS:

- To be used by dental professionals only.
- Wear gloves and protective eyewear.
- Advise patient not to smoke, consume highly coloured, hot or cold foods or beverages for at least 2 days.
- Keep out of reach of children.
- Not to be used by pregnant or lactating women or children under 14 years old.
- Anyone with a history of chemical allergies is advised to undergo allergy testing by a specialist before using this product.
- If patient experiences unusual sensitivity, remove gel and rinse affected area. Treat any post-treatment sensitivity with a desensitising gel or toothpaste.
- SD5 available at www.sdi.com.au or contact your regional representative.

FIRST AID:

- Eye (contact):** Hold eyelids open. Wash thoroughly with water for 15 minutes and seek medical attention
- Skin/Tissue (contact):** Wash thoroughly with water and seek medical advice if symptoms persist.
- Ingestion (large amount):** Drink plenty of milk or water and seek medical advice.
- Inhalation:** No symptoms expected.

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